

YOURmeds

Pharmacy Guide

Making A Pack


How to Make a Pack

For a video on how to make the pack please go to;

<https://marsoftware.sharepoint.com/:v:/g/shared/EYwftAm-Nz1Fh2yZEI86S-oBaSTN56QFFDC5epw6C3YHIg?e=AdQ1R0>

Sealing the Tray

1. Place the plastic blister into the black pharmacy platen and push out the section numbered 1. Throw this away (recycled)
2. Fold the plastic tongue down and away from you towards the platen, reinforcing the fold with your thumb and forefinger.

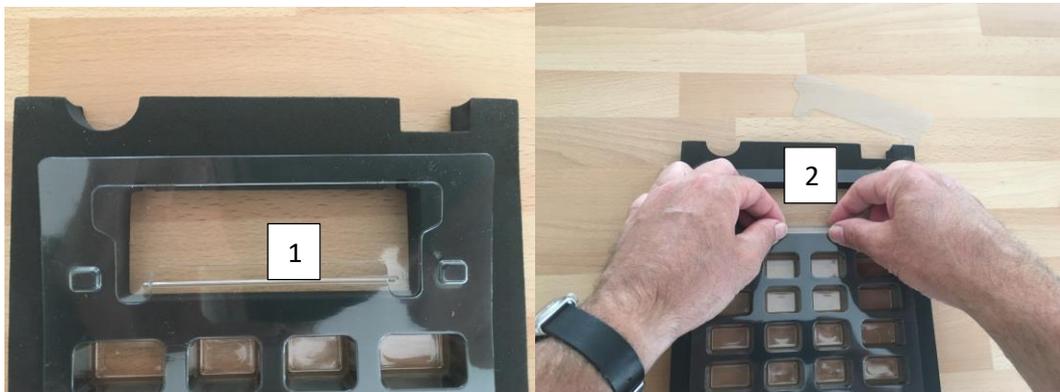

3. Dispense as usual.




4. Gently peel away the Top portion of the seal, as shown below.



5. Line the seal up to the guides on the plastic tray. These are shown as A and B below. Make sure to press firmly to stick the top of the seal on the plastic tray, and to ensure that the sides are aligned to A and B above
6. Peel the liner on the rear of the seal
- 7 As you peel the liner away from the seal (A), very lightly stick the seal over the blister cavities (B)
- 8 Once the whole backing A et is removed, gently smooth over the seal with your palm to finish sticking the seal down

NOTE: IF YOU SEE ANY CREASES IN THE SEAL, YOU WILL NEED TO PUT ON A NEW SEAL.

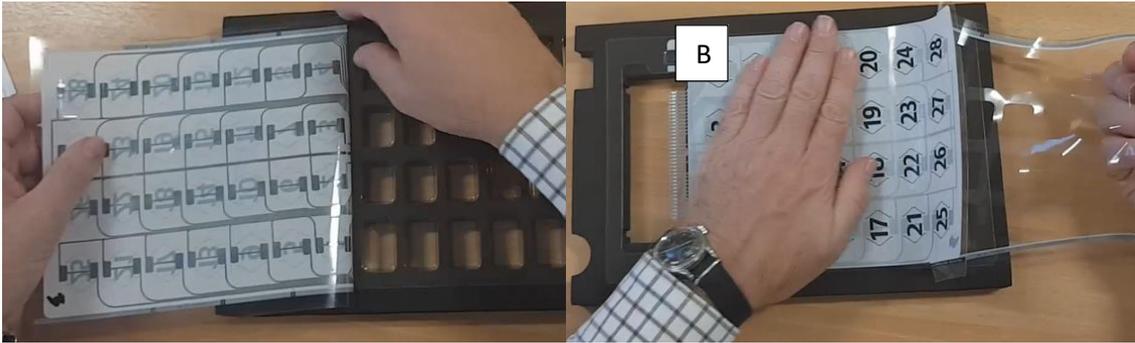

Making the Cover

1. Remove the liner from the blister card numbered 1 and position the card to the blister guides numbered 2 and 3 as illustrated. Once located stick the card firmly onto the seal.



2. Once the card is securely adhered to the blister and seal, fold the tongue of the seal down and away towards the platen, reinforcing the fold with your thumb and forefinger.



3. Remove completed blister card from the Black Pharmacy Platen and insert the patient white foam platen for sturdiness. Finally print a YOURmeds label from the YOURmeds MAR dispensing solution and stick to the inside page of the pack



Integrity Checks

- Every pack that is made **must** have its integrity checked. You do this by attaching your pharmacy test tag to the completed pack. When you attach a tag to a pack it will first need to stabilise, this will take 10-20 seconds and will display a message saying “Stabilising”, when it has finished it will say “Connected” (see images below). When the connected message appears, you will see a diagram of the pack on the left-hand side of the tag’s screen. The diagram shows open blisters as a black square and unopened blisters as a white square (see images below). To check the pack’s integrity, **you must ensure that every pack dispensed has no open blisters on the tag’s screen.**



YOURmeds Dispensing Standards

As a YOURmeds partner pharmacy **you must ensure that;**

- The lining covering the silver pins are correctly folded downwards or completely removed. This is critical for a YOURmeds tag to read a YOURmeds pack.
- A YOURmeds label is used on every YOURmeds pack.
- Every YOURmeds pack must have its integrity checked before being delivered.
- The patient’s tag is attached to the first pack before delivery.
- The correct tag is delivered to the correct patient as tags are preassigned using the ID number located on the back of the tag.